

VENDLET®

DIGNIFIED PATIENT HANDLING



WHAT IS A VENDLET?

VENDLET is an electrical turning device. VENDLET consists of a slide sheet, a turning sheet, a hand control and two motorised bars which are mounted on each side of the bed.

By pressing the hand control, the sheet is tightened around the bars, so the patient is moved or turned.

USE VENDLET TO

- Reposition the patient to lateral, prone or supine position for hygiene, examination and positioning purposes and applying slings etc.
- Move the patient from one side of the bed to the other
- Reposition the patient up in the bed
- Mobilize the patient in and out of the bed
- Transfer the patient from one bed to another

VENDLET MODELS

VENDLET V5: An electric turnsheet with a weight capacity up to 200 kg.

VENDLET V5+: An electric turnsheet with a weight capacity up to 400 kg.

TARGET GROUP

- Patients with few physical and / or cognitive resources and palliative patients
- Patients who receive care, hygiene, etc. in bed or are lifted in/out of bed
- Patients who are at risk of pressure ulcers since the uniform and quiet transfer in the sheet minimizes friction and shear, which can cause damage to tissues and lead to pressure ulcers
- Patients who suffer from spatial awareness since VENDLET often is seen to have a positive impact on this group of patients
- Bariatric patients

BEDS AND MATTRESS


VENDLET is designed for installation on care and hospital beds approved to EN1970, EN60601-2-38 or EN60601-2-52.

The bed should have a rectangular steel frame with a width of min. 20 mm, a length of min. 1950 mm and a height of 45-50 mm.

VENDLET works with all mattress types, including pressure-relieving mattresses. We recommend using mattresses with a firm side construction.

VENDLET®

DIGNIFIED PATIENT HANDLING

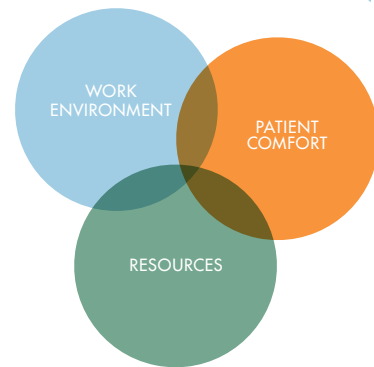

VENDLET IMPROVES THE WORK ENVIRONMENT BECAUSE:

- It reduces physical strain and heavy lifting of the patient for the caregiver
- It offers greater dignity and creates a feeling of well being for both the caregiver and patient

VENDLET INCREASES PATIENT COMFORT BECAUSE:

- The patient experiences a consistent and steady turning procedure, which is comfortable because the sheet supports the body in full length
- The caregiver can support the patient during the procedure and ensure a proper and comfortable position in bed
- The patient does not feel like a burden on the caregiver

VENDLET OFFERS AN EFFECTIVE USE OF RESOURCES BECAUSE:

- The caregiver in most cases can work alone with the patient, which simplifies the planning of the daily care and reduces staff requirements
- Using VENDLET leads to an easier and more efficient workflow, which leaves resources available for other purposes
- Using VENDLET reduces the number of work related injuries which reduces absenteeism

Technical information	VENDLET V5	VENDLET V5+
Item no.	6000000	6000021
Max. weight capacity	200 kg	400 kg
Supply voltage	230 V / 50 Hz	230 V / 50 Hz
Output voltage	24 V DC	24 V DC
IP class - Control unit, hand control, actuator	IPX6	IPX6
Compliance	European Medical Device Directive 93/42/EEC	European Medical Device Directive 93/42/EEC



All rights reserved.
Copyright 2014
Vendlet ApS

Distributor Australia:

HLS Healthcare
Head Office:
3/56 Norcal Road, Nunawading, VIC 3131
Tel. +61 3 9878 4448

admin@hlshealthcare.com.au
www.hlshealthcare.com.au

Manufacturer:

Vendlet ApS
Egelund 33, 6200 Aabenraa
Denmark
Tel. + 45 9631 0050

vendlet@vendlet.dk
www.vendlet.com


VENDLET
VALUE IN CARING