

VENDLET

V5 and V5+


QUICK GUIDE

WHEN VENDLET IS USED AS SIDE RAIL



The  button works as a stop button



1. The surface of the bed must be flat and horizontal.
2. Remove the plastic sides and hang them on the support bar.
3. Double-click the  button to activate the VENDLET system. The VENDLET system produces an acoustic signal and the green diode over the power symbol turns on. When the hand control has been inactive for 90 seconds, the system is off again.
4. The patient is repositioned by operating the hand control.

Tighten the sheet on the red bar



Slacken the sheet on the red bar



Raise the red bar



Lower the red bar



Power w. illuminated diode/ stop



Tighten the sheet on the blue bar



Slacken the sheet on the blue bar



Raise the blue bar



Lower the blue bar



Under bed light (accessory)

5. After repositioning the patient; raise the bars if these have been lowered.
6. Attach the plastic sides – 2 on each side. See also the side rail guide.



During the procedures always stay focused on the patient.

VENDLET must be used and installed in accordance with the manual

Scan the QR code to watch instruction videos




VENDLET

V5 and V5+

QUICK GUIDE

WHEN VENDLET IS NOT USED AS SIDE RAIL

1. The surface of the bed must be flat and horizontal.
2. Raise the bars on both sides.
3. Double-click the  button to activate the VENDLET system. The VENDLET system produces an acoustic signal and the green diode over the power symbol turns on. When the hand control has been inactive for 90 seconds, the system is off again.
4. The patient is repositioned by operating the hand control.



The  button works as a stop button

Tighten the sheet on the red bar ←
Slacken the sheet on the red bar ←
Raise the red bar ←
Lower the red bar ←
Power w. illuminated diode/ stop ←



→ Tighten the sheet on the blue bar
→ Slacken the sheet on the blue bar
→ Raise the blue bar
→ Lower the blue bar
→ Under bed light (accessory)



5. The bars must be lowered on both sides before the patient is left unattended in the bed.



During the procedures always stay focused on the patient.

VENDLET must be used and installed in accordance with the manual

Scan the QR code to watch instruction videos

